**Intended use****Intended purpose**

Access to the oral cavity of the patient

**Use**

For dental use only.

**Description**

OptraGate® is an auxiliary aid for clinical use which allows the working field to be accessed with ease.

It provides retraction of lips and cheeks during dental treatment, offers the clinician increased visibility and accessibility and facilitates saliva and moisture control in the oral cavity. The device, which features three-dimensional flexibility and elasticity, ensures patient comfort and assists in keeping the patient's mouth open. OptraGate is available in sizes Regular, Small as well as Junior for children between 5 and 10 years of age.

**Indications**

Retraction of the lips and cheeks during diagnostic, preventive and therapeutic dental procedures as well as orthodontic procedures: e.g. diagnostics, professional teeth cleaning, fissure sealing, tooth whitening, periodontal treatment, restorative procedures, cavity preparation, impression taking and cementation of restorations and orthodontic retention elements.

**Contraindications**

The use of OptraGate is contraindicated

- in oral surgical and implantological procedures;
- if the patient is known to be allergic to any of the components of OptraGate.

**Limitations of use**

-  OptraGate is for single use only. The product is not intended to be reprocessed or reused. The material is not suitable for sterilization. Cleaning and disinfecting agents may have a detrimental effect on the material and lead to undesirable reactions in the patient.
-  OptraGate is not sterile and thus not suitable for use in oral surgical or implantological procedures.

**Side effects**

There are no known side effects to date.

**Composition**

Thermoplastic elastomer (SEBS) and polypropylene. OptraGate is latex-free.

**Application**

The elastic material of OptraGate is stretched by means of two rings. The thicker ring of the device is positioned within the gingivobuccal fold, while the thinner ring, which features two tabs, remains outside the mouth. The elastic component between the two rings thus embraces the lips of the patient and provides retraction due to the restoring force of the rings (see Figure 1).

Fig. 1: OptraGate in place with the tabs pointing downward. It does not interfere with the patient's occlusion.

1. Select the appropriate size of the OptraGate. The selection of the appropriate size should be guided by the distance between the corners of the mouth when the mouth is in a relaxed position (i.e. with the lips closed or slightly open). In order to obtain sufficient retraction of the oral soft tissues, the diameter of the inner ring should ideally be approx. 2 cm wider than the mouth corner distance. Therefore the following guidelines apply:

- Size Junior: Mouth corner distance  $\approx$  4.5 cm
- Size Small: Mouth corner distance  $\approx$  6 cm
- Size Regular: Mouth corner distance  $\approx$  7 cm

The orientation scale with the respective guide marks imprinted on the foil bag can be used as an aid in the selection of the correct size. It is important not to use a rigid orientation aid for the determination of the mouth corner distance, but to follow the slightly curved course of the lips. In case of doubt, we recommend using the larger size in order to

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ensure a better fit. The size (R for regular, S for small or J for Junior) is indicated on the packaging as well as on the left tab of the OptraGate. In order to ensure optimum function, we recommend inserting the largest possible size that fits the patient's mouth.

- OptraGate may be inserted from both the 12 o'clock position with the patient lying down, or from the 8 o'clock position with the patient sitting upright (or, the 4 o'clock position, if the clinician is left-handed).
- To ease placement, with the tabs pointing downward, the operator should grasp the thicker intraoral ring by reaching through the thinner extraoral ring. The inner ring should be held between the thumb and middle finger and pressed together slightly (see Figure 2).

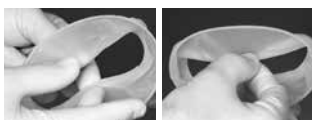
Fig. 2: Correct hold on the intraoral ring with thumb and middle finger. The intraoral ring is slightly compressed in the process.

- The slightly compressed intraoral ring is then inserted into the buccal corridor on one side, so that the elastic component embraces the corner of the mouth and the outer ring is still located extraorally (see Figure 3).

Fig. 3: The thicker intraoral ring is positioned between the teeth and the corner of the mouth.

- As soon as the OptraGate is securely in place on one side, the other side is inserted in the same manner by slightly bending it (see Figure 4).

Fig. 4: The intraoral ring is positioned behind the two corners of the mouth.

- Subsequently, the intraoral ring is placed behind the lower and upper lips and thus OptraGate assumes its final, stable position (see Figure 5). The intraoral ring can be inserted with more ease if the patient's mouth is relaxed.

Fig. 5: Final positioning behind the lower and upper lip.

If, in individual cases, the intraoral ring should show a tendency to slip out of the gingivobuccal fold when the mouth is closed completely, positioning of the intraoral ring deeper inside the vestibule is usually sufficient to solve the problem. Sometimes, the use of another size may also be a solution. The securely positioned OptraGate improves the overall view and accessibility of the working field. Lateral movements of the mandible and opening of the mouth to various degrees may provide additional space and access to the treatment area. The OptraGate may even be left in place when checking the occlusion.

- To remove OptraGate, grasp the upper area of the extraoral ring with the help of a paper towel and pull it slightly downwards so that the intraoral ring is loosened from the upper gingivobuccal fold (Figure 6). The intraoral ring can then be easily removed from the lower gingivobuccal fold and hygienically disposed of using the paper towel.

Fig. 6: Removal of the OptraGate

#### Safety information

- In the case of serious incidents related to the product, please contact Ivoclar Vivadent AG, Bendererstrasse 2, 9494 Schaan/Liechtenstein, website: [www.ivoclarvivadent.com](http://www.ivoclarvivadent.com), and your responsible competent authority.
- The current Instructions for Use are available in the download section of the Ivoclar Vivadent AG website ([www.ivoclarvivadent.com](http://www.ivoclarvivadent.com)).

#### Warnings

During bleaching procedures, care should be taken to prevent any bleaching agent from getting trapped between the OptraGate and the soft tissue in order to prevent irritation of the skin and mucous membrane.

#### Disposal information

- Remaining stocks must be disposed of according to the corresponding national legal requirements.

#### Shelf life and storage

- Storage temperature 2–28 °C (36–82 °F)
- Keep away from sunlight
- Do not use the product after the indicated date of expiration.
- Expiration date: See note on packaging

#### Additional information

Keep material out of the reach of children!

The material has been developed solely for use in dentistry. Processing should be carried out strictly according to the Instructions for Use. Liability cannot be accepted for damages resulting from failure to observe the Instructions or the stipulated area of use. The user is responsible for testing the products for their suitability and use for any purpose not explicitly stated in the Instructions.

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**Ivoclar Vivadent Marketing Ltd.**
**Ivoclar Vivadent AB**
**Ivoclar Vivadent Limited**
**Ivoclar Vivadent, Inc.**